

Aptima[®] Urine Specimen Collection Procedure




Aptima Combo 2[®]
Assay for CT/NG




Overfilling or underfilling will require a new sample collection tube to obtain a valid result.

Transfer urine into collection tube.


Fill between black lines only.

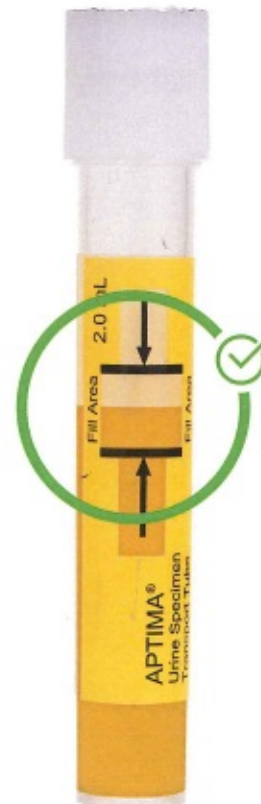

ACCEPT

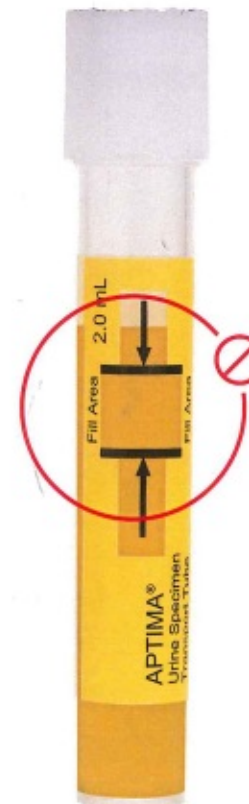

**OVERFILLED
REJECT**

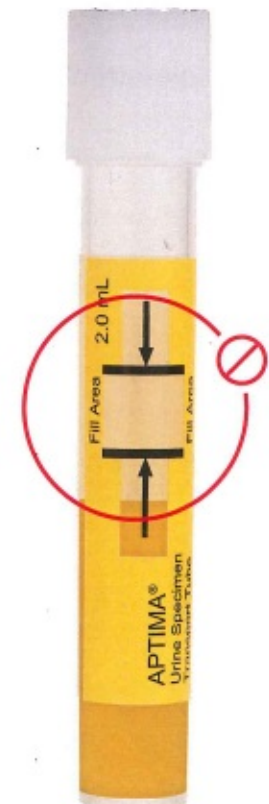

**UNDERFILLED
REJECT**

Refer to the geography-specific package insert for specific sample type availability.

* The Aptima Zika Virus assay:

- This test has not been FDA cleared or approved;
- This test has been authorized by FDA under an EUA for use by authorized laboratories;
- This test has been authorized only for the detection of RNA from Zika virus and diagnosis of Zika virus infection, not for any other viruses or pathogens; and
- This test is only authorized for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of in vitro diagnostic tests for detection of Zika virus and/or diagnosis of Zika virus infection under section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.